

How to Build a Moses Trial Device

Using a Moses Bite Stix to try a vertical and protrusion setting you determine; you will be hand crafting using impression material and a surgical blade. We recommend [Kettenbach Futar](#). The final step is trimming with a blade to create the Moses Trial Device.

1

Determine trial mandibular position

This is your professional determination of where the mandible should be for optimum airway stenting



2

Build the Moses Trial Device

- Inject Bite registration material to hand craft a Moses Alpha



3

Trim the Moses Trial Device

- Remove bite tool and create open anterior
- Trim the trial device for overnight usage
 - Should look like a Moses Alpha

